

Scrotal Shield

Helps prevent scrotal tissue from being drawn into the vacuum cylinder



INSTRUCTIONS FOR USE OF THE SCROTAL SHIELD:

Prior to using the vacuum therapy system, insert penis through the hole in the center of the scrotal shield and slide the shield to the base of the penis, laying flat against your pubic area. Place the open end of the vacuum therapy system up against the scrotal shield and activate the system.

After an erection is formed and the tension ring is in place at the base of the penis, remove the vacuum therapy system, then remove the scrotal shield by pulling the scrotal shield over the handles of the tension ring and off of the penis.

Order online at: shopping.rejoynmedical.com

Or call 866-374-7640

rejoyn
Medical Systems, Inc.